


<b>Kit number: XXXXX</b>	<b>ID subject: XX/XXXX</b>	
	<b>Mod. XXX-XXX KIT USAGE INSTRUCTIONS</b>	Atlanta Tech Park 107 Technology Parkway PEACHTREE CORNERS, GA 30092 E-Mail: <a href="mailto:info@magisnat.com">info@magisnat.com</a>

Dear Customer,

Thank you for choosing the MAGISNAT DNA Wellness Test. This kit usage instructions document will guide you through the process of collecting your biological sample using the buccal swab provided in the kit. Please read the following instructions carefully before proceeding.

### Kit Number

This is your kit number: XXXXXXXX

Please check if it matches the kit number provided to you at the time of purchase.

### Kit Contents

This biological sampling kit contains:


- Two buccal swabs
- Two collection vials
- The return envelope
- Kit instructions

Please check and, if something is missing, promptly inform us.

### Step-by-Step Sampling Instructions

Please, carefully read and follow these instructions for a correct sampling:

1. Place the unopened swabs and vials on clean, levels surface.
2. Remove one swab from its package and firmly scrape the inside of one cheek while rotating the swab for 45 seconds. Note: do not eat, drink, or chew gum for 60 minutes prior swabbing your cheeks.
3. Open one vial, insert the swab with the tip down, align the black line on the swab to the edge of the vial and snap it off with the swab tip remaining in the vial's liquid. Note: if the liquid from the vial is spilled, please send in you vial anyway as the sample might still be viable.
4. Use the second swab and vial to repeat the points 2 and 3 on the opposite cheek.

Kit number: XXXXX	ID subject: XX/XXXX	
	<b>Mod. XXX-XXX KIT USAGE INSTRUCTIONS</b>	Atlanta Tech Park 107 Technology Parkway PEACHTREE CORNERS, GA 30092 E-Mail: <a href="mailto:info@magisnat.com">info@magisnat.com</a>

5. Ensure that the cap is securely fastened on both vials. Insert both into the plastic bag and then seal the bag.

### **Packaging and Shipping:**


For correct packaging and shipping:

1. Place the sealed plastic bag in the return envelope we provided and seal the envelope.
2. Send the envelope with your biological samples. The return postage is prepaid.

### **Further recommendations and reminders:**

Please, remember to access your reserved area and sign the informed consent. The system uses a handy and straightforward system for digital signature via OTP (One Time Password). If you don't sign this document, we are unable to process your collected samples.

We strongly invite you to fill also the optional questionnaire, so that we can provide you with some more insights and advice about your lifestyle. Moreover, you will give a priceless contribution to our research in the field of genomics. As a thank you, you will receive a 10% discount for purchasing dietary supplements in our e-commerce.

<b>Kit number: XXXXX</b>	<b>ID subject: XX/XXXX</b>	
	<b>Mod. XXX-XXX</b> <b>KIT USAGE</b> <b>INSTRUCTIONS</b>	Atlanta Tech Park 107 Technology Parkway PEACHTREE CORNERS, GA 30092 E-Mail: <a href="mailto:info@magisnat.com">info@magisnat.com</a>

## DISCLAIMERS

All information regarding the DNA Wellness Test is provided in good faith. However, we make no warranty of any kind, expressed or implied, as to the accuracy, adequacy, validity, reliability, or completeness of the information regarding the DNA Wellness Test.

While we have made every attempt to ensure that the information contained in this DNA Wellness Test is accurate to the best of our knowledge, we are not responsible for any errors or omissions or for the results obtained from the use of this information.

**The final results obtained by the Low-Risk General Wellness Test have not been evaluated by the Food and Drug Administration, and they are not intended to diagnose, treat, cure, or prevent any disease.**

Before taking any action based on the information provided by the DNA Wellness Test, we urge you to consult with appropriate professionals as it is not a substitute for professional medical advice. In any case, we are not liable if you receive inadequate or even dangerous advice or recommendations for your health from third parties.

Genetic test results can have psychological implications, so it's important to be prepared for potential emotional distress or anxiety related to learning about health risks.

The use, any losses and/or damages incurred because of the use of the DNA Wellness Test, and the reliance of any information contained in this DNA Wellness Test are solely the responsibility of the user.

Any testimonials regarding the DNA Wellness Test are personal and are not representative of all users. We do not claim, and you should not assume that all users have the same experiences.

We make every effort to ensure the highest standards, we collaborate with a CLIA-certified laboratory (Clinical Laboratory Improvement Amendments) and have validated the process to the best of our abilities. However, genetic tests inherently have limited sensitivity and specificity, albeit high. As a result, different tests may yield partially different results, also due to technical details. We do not assume any responsibility if such events were to occur.

The considered polymorphisms are analyzed solely for the purposes of the present test. There may be other correlations, known or unknown, that have not been reported. Such omission, whether intentional or not, should not be considered a fault if it occurs.

Some analyzed variants may be interpreted differently in other contexts, potentially leading to genetic discrimination. We are not responsible for this.

The data collection and processing system is secure, and the DNA sample is discarded 180 days after the analysis. We are not liable for any data breaches resulting from cyber-attacks or rare events beyond the control of our standard security measures.

Genetic test results can have psychological implications, such as potential emotional distress or anxiety related to learning about health risks. We are not responsible for this.

If you have any questions, concerns, or need support in understanding the test, please call our support team on +XXXXXXXXXXXX or email [info@magisnat.com](mailto:info@magisnat.com) or visit our website [www.magisnat.com](http://www.magisnat.com).

Kit number: XXXXX

ID subject: XX/XXXX



**Mod. XXX-XXX  
KIT USAGE  
INSTRUCTIONS**

Atlanta Tech Park  
107 Technology Parkway  
PEACHTREE CORNERS, GA  
30092  
E-Mail: [info@magisnat.com](mailto:info@magisnat.com)